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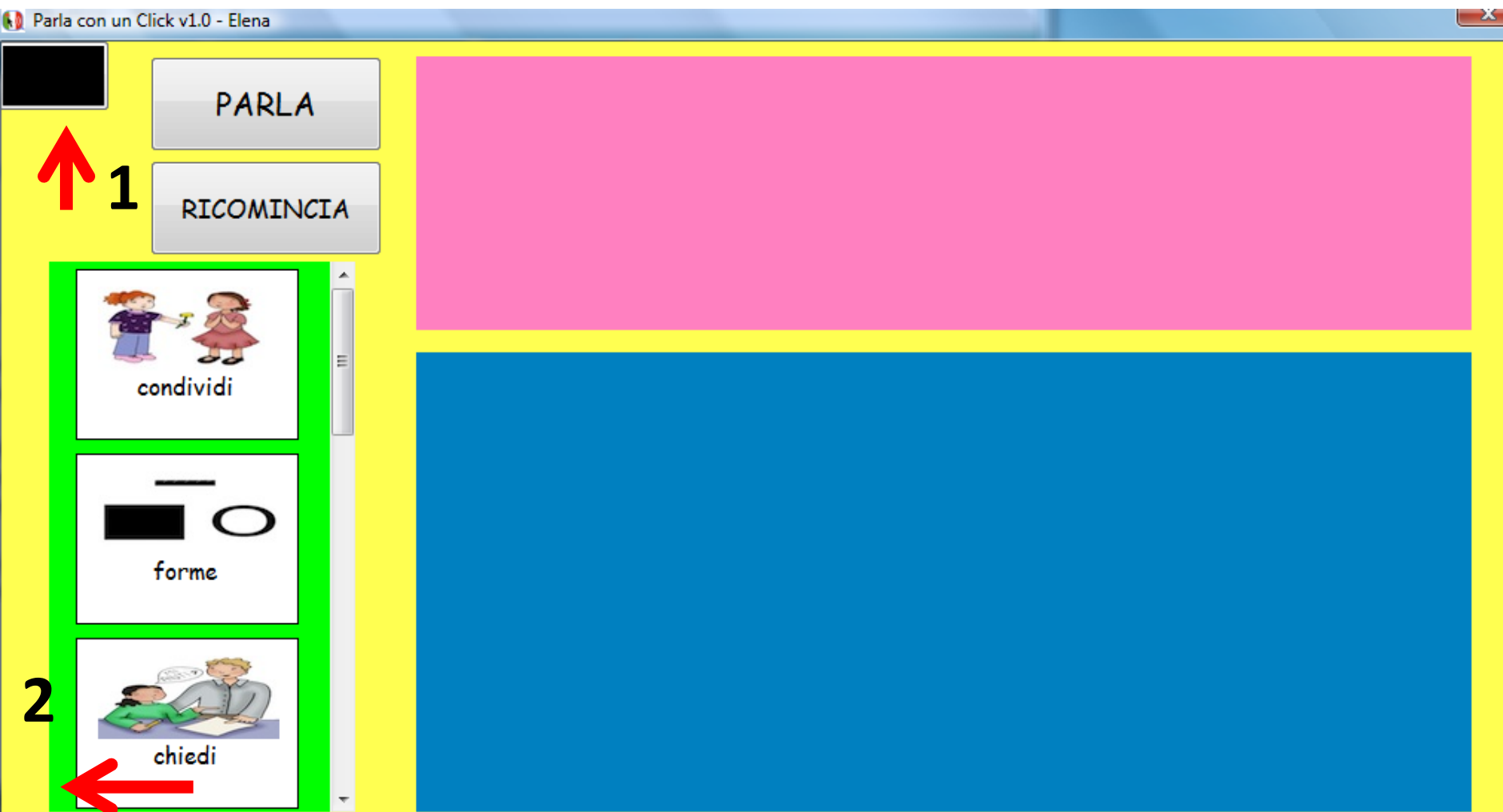
The image shows a sequence of three screenshots from the 'Parla con un Click v1.0' software interface, illustrating the steps to insert an image. The interface is divided into three phases:

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Red arrows and numbers 1 through 6 indicate the sequence of actions. Arrows 1-4 point to the buttons in Fase 1. Arrow 5 points to the 'Ridimensiona automaticamente' button in Fase 2. Arrow 6 points to the 'Modifica Immagine' button in Fase 4. Blue arrows indicate the flow from Fase 1 to Fase 2, and from Fase 3 to Fase 4.

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
At the bottom left, it says 'Designed and Developed by: TOOKTY LLC. Norfolk VA USA'. At the bottom center, there is a URL: <http://www.tookty.com>.

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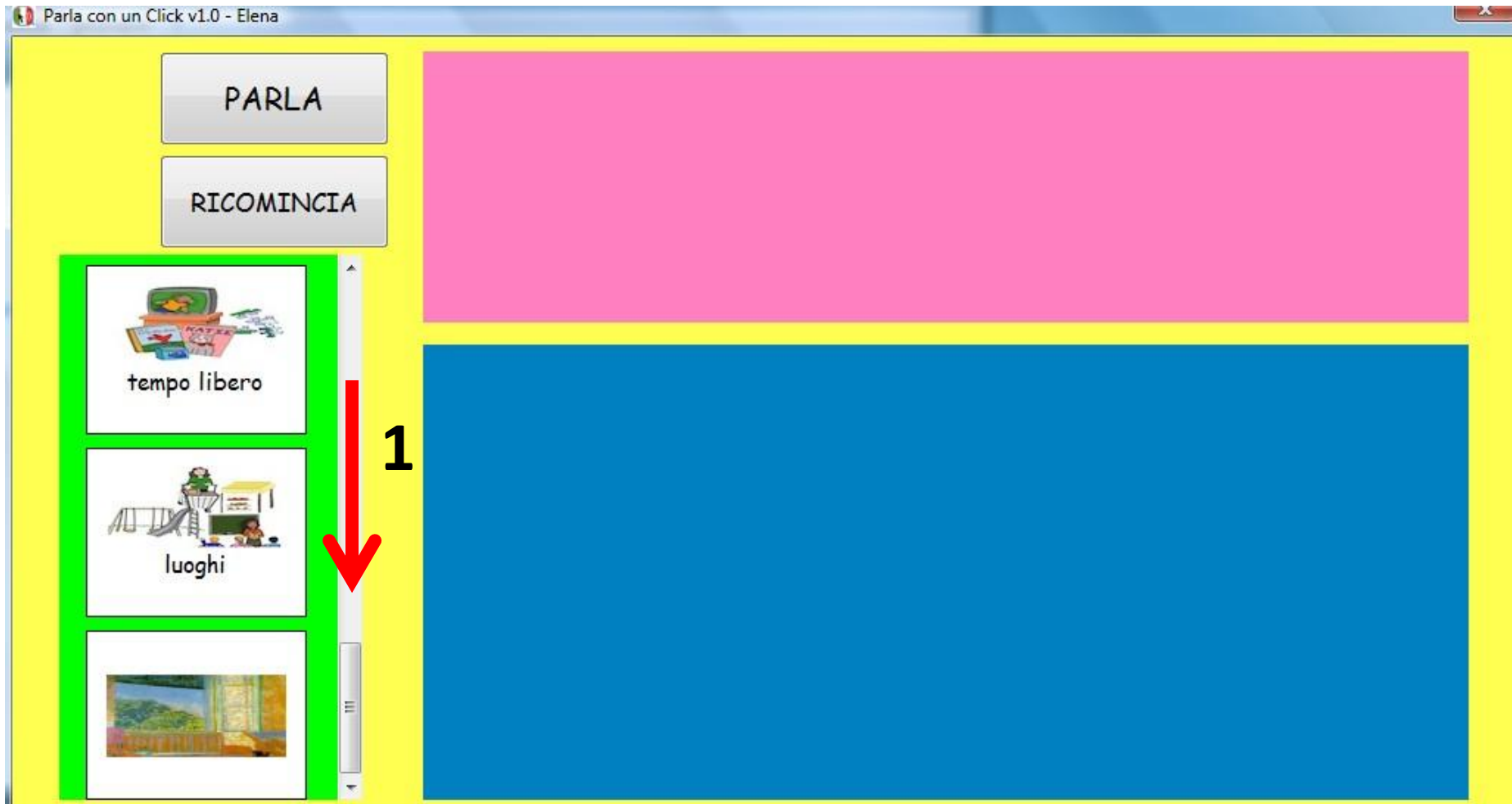The screenshot displays the 'Parla con un Click v1.0 - Elena' software interface, illustrating the process of inserting an image. The interface is divided into three main phases:

- Fase 1: Crea immagine** (Create image): This phase includes buttons for 'Niente da incollare', 'Carica', 'Cattura da webcam', and 'Crea da testo'. A red arrow labeled '1' points to the 'Inserisci nuova immagine' button in the main window, and red arrows labeled '2', '3', and '4' point to the 'Carica', 'Cattura da webcam', and 'Crea da testo' buttons respectively.
- Fase 2: Modifica immagine** (Modify image): This phase includes buttons for 'Ridimensiona automaticamente', 'Ritaglia', and 'Ripristina'. A red arrow labeled '5' points to the 'Ridimensiona automaticamente' button. Below these buttons is a preview of a painting (Pablo Picasso's 'Guernica') with percentage options (50%, 75%, 90%) for automatic resizing.
- Fase 3: Anteprima immagine** (Image preview): This phase shows a small thumbnail of the painting. A downward arrow indicates the transition to the next phase.
- Fase 4: Conferma modifiche** (Confirm changes): This phase includes a 'Modifica Immagine' button. A red arrow labeled '6' points to this button.

At the bottom left, the text reads: 'Designed and Developed by: TOOKTY LLC'.

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The screenshot shows the 'Parla con un Click v1.0' software interface. The window title is 'Parla con un Click v1.0 - Elena'. The interface is divided into several sections:

- Top Bar:** Contains an 'Esci' button.
- Navigation:** Includes 'Indietro' and 'Inserisci nuova immagine' buttons.
- Image Preview:** A small thumbnail of a landscape image is shown in the top right.
- Image Settings:** A larger image of a person is shown in the center. To its left are a 'Visibile' checkbox (checked), 'Tu=0; Elena=0', an 'Ordine' dropdown (set to '01'), and an 'Elimina Immagine' button.
- Actions:** Buttons for 'Cambia Immagine', 'Copia Immagine', and 'Registra audio' are present. A red arrow labeled '3' points to the 'Registra audio' button.
- Text and Border Settings:** A 'Cambia testo' button is followed by a text input field (with a red arrow labeled '1' pointing to it). Below it is a 'Tipo Bordo' dropdown menu (set to 'Nessun bordo', with a red arrow labeled '2' pointing to it) and a 'Nessuna Posizione' dropdown menu.
- Bottom Bar:** Contains an 'Indietro' button and a large red arrow labeled '4' pointing to a 'Conferma Impostazioni' button.

At the bottom of the window, the text reads: 'Designed and Developed by: TOOKTY LLC. Norfolk VA USA' and 'http://www.tookty.com'.

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